

A-dec 6300 Patient Light

Operation

Never operate an Ultrasonic Cleaner without:

- Focus light pattern by turning large slotted screw on back of light head
- Adjust intensity (high, medium, low); utilize lowest comfortable intensity setting

Maintenance – (as needed)

- Clean reflector and shield with damp lint free cloth / gauze
- Replace bulb when cool; extra bulb located at light head
 - do not touch bulb with bare hands
 - do not wiggle bulb in or out, push straight in or pull straight out
- Check toggles that hold shield in place for proper attachment